Index

1.	NL – Stappenplan installatie CR2 AED	3
2.	UK – Roadmap LIFEPAK CR2 AED	5
3.	PT – Instalação – DAE LIFEPAK CR2	7
4.	ES - Instalación– DEA LIFEPAK CR2	1
5.	IT – Procedura di installazione DAE CR2	3
6.	PL– Instrukcja instalacji LIFEPAK CR2 AED	6
7.	FR – Instructions pour DAE LIFEPAK CR2	8
8.	DE – Schritt-für-Schritt-Plan für die Installation CR2 AED	10

1. NL – Stappenplan installatie CR2 AED

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2. UK – Roadmap LIFEPAK CR2 AED

For detailed information on the LIFEPAK CR2 AED, please refer to the Owners's Manual. This is included with the AED and can also be found at cprmanual.com

Steps	Instructions
Prior to installation	1. Install the battery in the AED. 2. The defibrillator will automatically start the selftest. Wait for 3 audible tones to sound before proceeding to the next step. <i>The selftest can take up to 4 minutes. Do not open the lid until you have heard the 3 tones otherwise, the selftest will not be completed.</i> 3. Verify that the green light at the top of the AED is flashing. <i>The status indicator will flash every 6 seconds through a hole on the top of the lid. A flashing status indicator means the CR2 is ready for use.</i> 4. Open and close the lid of the AED. Make sure you have the following WiFi network data available: Networkname: WiFi network password: Remove the USB cable from the box

Step 1	You will receive an e-mail from LIFELINK with a link to log in to the AED program. 1. Press on the link. 2. If you haven't received an e-mail, contact your supplier.
Step 2	Download and install the WiFi configuration utility on your PC 1. Select Resource Center from the left menu. 2. Select WiFi configuration Tool (WiFi configuration program) 3. Click the icon ⓘ In the top right corner and press on 'Run'. (You will now download the WiFi configuration program). 4. Accept the WCT.xxxxx_Setup.exe file and install the program. You will see the InstallShield Wizard, select English and press 'Next'. 5. Agree to the terms and conditions and press 'Install' 6. You will see: Start WiFi-Configuration Tool – Press 'Finish'.
Step 3	Open the WiFi setup utility (It will not open automatically) and follow the on-screen instructions 1. In 'Registration', select your country 2. Enter your username and password. The username is your e-mail address. The password you created yourself in step 1. 3. You will now be instructed to 'Connect LIFEPAK CR2 to PC'. Use the USB-cable from the box and follow the on-screen instructions. 4. In the screen 'WiFi network settings', choose your own network and enter the WiFi password. 5. Press 'Next'. 6. You will now see 'Update successful'. 7. Press 'Next', disconnect the USB cable and press 'Close'
Step 4	Bring the AED to the site and test the wireless connection 1. Open the AED lid at this location. 2. Wait for the spoken instructions to start. 3. Immediately press and hold the buttons 'LANGUAGE AND CHILD MODE' simultaneously (outer 2 buttons). 4. You will hear 'Device ready'. 5. Close the lid. 6. Place the AED in the predetermined location (AED will now connect).

	If you hear: 'WiFi connection initiated' – go to step 5. If you hear: 'Mobile connection (3G) initiated' – go to step 6.
Step 5	Check the WiFi connection
	You hear: 'WiFi connection initiated' > The connection is underway (<i>This will take some time</i>) You hear: 'Connection established' (This can take several minutes). The AED will now emit a signal every few seconds. Upon completion, you will hear 'Communication complete'. Turn off AED with WiFi only. ➤ No 3G – continue to step 7. ➤ 3G – Continue to step 6.
Step 6	Confirm mobile connection (3G), if applicable.
	The AED will automatically connect to the mobile network (3G) You hear: 'Mobile connection initiated'. Followed by: 'Connecting' (This takes some time). You hear: 'Connection established'. After all updates are complete, you will hear: 'Communication completed' followed by 'Shutdown'.
Step 7	Return to you PC
	1. Log in to your account (see step 1). 2. Go to page 'Equipment'. 3. Verify that your AED is ready and is shown in green
Step 8	The wireless setup is now complete
	For information or assistance, you can reach us at the details below.
Step 9	Move AED to correct location
Multiple locations	1. Go to 'Locations' 2. Press the '+' sign 3. Enter the details of the location where the AED is hanging 4. Return to 'account' 5. Press 'equipment' uitrusting 6. Press  device transfer 7. Select the serial number you want to move to another location 8. Press the check mark in the top right corner 9. Select the correct location and press 'Transfer'

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